

Urin analyse / urin analysis

Hænder vaskes og udstyret findes frem:
Wash your hands and find the equipment:



Dyp testen i urin så alle de farvede felter er i kontakt med urin:
Dip the test in urine so all the colored fields are covered:



VEND/TURN →

Aflæs farvekoden ved at holde testen vinkelret hen til beholderen, og rapporter resultatet til Radio Medical:

Read the color code by holding the test to the box and rapport the result to Radio Medical:



Eksempler / Examples:

3+ for blod / 3+ for blood

4+ for glucose / 4+ for glucose

